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
Geysir CDD 1050
model Basic

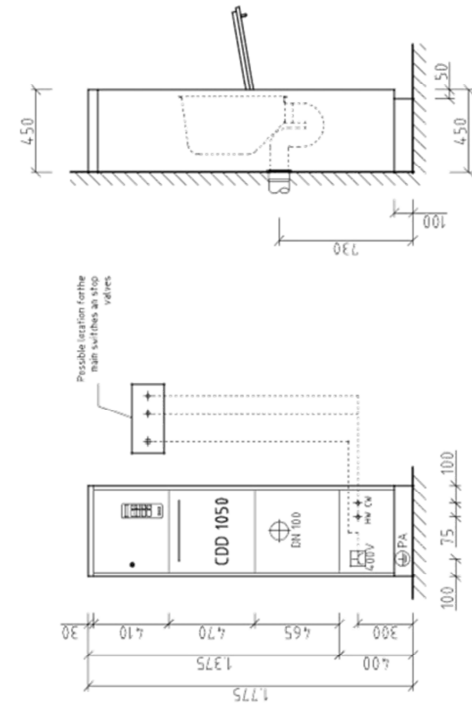Technical data

provided by the device: Media connections wall fitting

CW

Cold water connection, recommended 10"4H, 18l/min, 2-5 bar
provided by the device: R 3/4" tube, 1,2 m in from the device
With union nut, angled filter
stop union as ball valve with 3/4" AG flat sealing

HW

Hot water connection, recommended 10"4H, 18l/min, 2-5 bar, min.40°C
provided by the device: R 3/4" tube, 1,2 m in from the device
With union nut, angled filter
stop union as ball valve with 3/4" AG flat sealing

Electrical connection:

provided by the device: cable on device 5x1,5mm², H05-RN, length=0,8m

Site: connected load: 4,00V, 3Ph, N/PE, 50Hz, 4, 85kW
all poles main switch and device junction box, security 10A

Possible alternative:

Site: connected load: 1x 230V, 1 P+N+E, 50Hz, 3,35 KW
all poles main switch and device part box, hedging 16A


DN 100

provided by the device: sifon with coupler coupling sleeve DN 100 / NW 110
connecting sleeve DN 100 / NW 110, effluent
ca. 60 °C, pH-value 5-9, downpipe rearwards


PA

potential equalization

water supply
(no scale)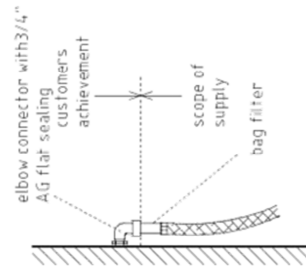

accomplishment
AISI 304

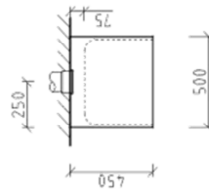General information:

Water Connections: Armored tubes incl. bag filter are included
Protective device, for example, electrical insurance, safety valves,
water filters 100 µm are on site
Stop valves and main switches are on site at user-friendly body.

This drawing is the property of the company Lischka GmbH and only with their consent totally or partially reproduced, copied or be given to third parties. Subject to change.	release principal	signature	date	signature	description: Geysir CDD 1050	project-number: construction drawing drawing-number: Date
	release LISCHEMA GmbH	signature	date	signature		
B	Elimates floor drain	20.12.2011	20.12.2011	Mr. van Busch	CDD 1050 - S	03.01.2011
A	Correcting the connections changes	30.05.2011	30.05.2011	Mr. van Busch	1.25 to DN A3	
	index:			name:		